



ACTION		
Undo	Control + Z	Command + Z
Redo	Control + Y	Command + Y
Copy Selected Contour	Control + C	Command + C
Copy All Contours	Control + A	Command + A
Paste Contour	Control + V	Command + V
Contours On/Off	Control + Shift + C	Option + Command + C
Delete All Contours	Control + Backspace	Command + Delete
Delete All Contours (Slice)	Control + M	Command + M
Delete All Contours (Phase)	Control + P	Command + P
Delete Selected Contour	Control + X	Command + Z
Delete Selected Contour (Slice)	Control + D	Command + D
Delete Selected Contour (Phase)	Control + O	Command + O
Forward All Contours (Slice)	Control + E	Command + E
Forward Selected Contour (Slice)	Control + F	Command + F
Select a Contour & Jump to the Next Contour	Tab	Tab
Hide All Contours	Shift	Shift
Smooth Selected Contour	Control + S	Command + S
Round Selected Contour	Control + L	Command + L
Refinement of Selected Contour	RMB Hold	RMD Hold
Click-Draw Contour	C + LMB Click	C + LMB Click
Threshold Segmentation Contour Definition	Alt + LMB Drag	Alt + LMB Drag
Drag Selected Contour	D + LMB Drag	D + LMB Drag
Push Selected Contour	P + LMB Drag	P + LMB Drag
Reposition All Contours	Control + LMB Drag	Command + LMB Drag
Default Window	Control + W	Command + W
Auto Windowing (Type I)	1	1
Auto Windowing (Type II)	2	2
Auto Windowing (Type III)	3	3
Auto Windowing (Min Max)	0	0
Change from Zoom to Phase Navigation	Shift + Scroll	Shift + Scroll
Hide cvi ⁴²		Command + H
Hide Others		Shift + Command + H
View All Images in Multiview	Space Bar	Space Bar
Add to Case Review	Control + R	Command + R
Refresh Study List	F5	Command + F5
Full Screen	F11	Command + F11
Preferences	F12	Command + " , "
Quit	Control + Q	Command + Q

Need help with cvi⁴²? Contact us!



www.circlecvi.com



bv_support@circlecvi.com (Europe Only)
support@circlecvi.com (Everywhere Else)



+1 403 338 1870



FAX +1 403 338 1895 (Worldwide)



Circle Cardiovascular Imaging
815 8th Avenue SW
Suite 250
Calgary, Alberta
Canada
T2P 3P2