



BASICS

Undo	+	Refresh Study List	
Redo	+	Add to Case Review	+
Delete Selected		Contours On/ Off	+ +
Copy Selected	+	Zoom to Phase Navigation	+
Copy All	+	Fullscreen	+
Paste	+	Preferences	+
Hide All		View All Images in Multi-view	
Play Cine		Go to Next/Previous Series	/

WINDOW

Default	+
Auto (Type 1)	
Auto (Type 2)	
Auto (Type 3)	
Auto (Min Max)	
Hide cvi42	+
Hide Others	+ +

CONTOURS

DELETE

Delete All	+
Delete All (Slice)	+
Delete All (Phase)	+
Cut Selected	+
Delete Selected (Slice)	+
Delete Selected (Phase)	+

FORWARD

MOVE

Forward All	+
Forward Selected	+
Select and Jump to Next	
Drag	+
Push	+
Reposition All	+

EDIT

Smooth	+
Round	+
Refine	hold
Click-Draw	+
Threshold Contour Definition	+